

Processing Type:

Deleted = Information is removed

Anonymous = Content is replaced with "anonymous"

0 = Content is replaced with "0"

Strict = Second option when installing Local/Anonymization Uplink

Non-strict = Third option when installing Local Anonymization Uplink

| DICOM Element | STRICT ANON Processing Type | STRICT or NOT_STRICT |
|---|------------------------------------|-----------------------------|
| RETIRED_AcquisitionComments | DELETE | STRICT |
| AcquisitionContextSequence | DELETE | STRICT |
| AcquisitionDeviceProcessingDescription | DELETE | STRICT |
| AcquisitionProtocolDescription | DELETE | STRICT |
| ActualHumanPerformersSequence | DELETE | STRICT |
| AdditionalPatientHistory | DELETE | STRICT |
| AdmissionID | DELETE | STRICT |
| AdmittingDiagnosesCodeSequence | DELETE | STRICT |
| AdmittingDiagnosesDescription | DELETE | STRICT |
| Allergies | DELETE | STRICT |
| RETIRED_Arbitrary | DELETE | STRICT |
| AuthorObserverSequence | DELETE | STRICT |
| BranchOfService | DELETE | STRICT |
| CassetteID | DELETE | STRICT |
| CommentsOnThePerformedProcedureStep | DELETE | STRICT |
| ConfidentialityConstraintOnPatientDataDescription | DELETE | STRICT |
| ContentCreatorIdentificationCodeSequence | DELETE | STRICT |
| ContributionDescription | DELETE | STRICT |
| CountryOfResidence | DELETE | STRICT |
| CurrentPatientLocation | DELETE | STRICT |
| RETIRED_CurveData | DELETE | STRICT |
| RETIRED_CurveDate | DELETE | STRICT |
| RETIRED_CurveTime | DELETE | STRICT |
| CustodialOrganizationSequence | DELETE | STRICT |
| DataSetTrailingPadding | DELETE | STRICT |
| DerivationDescription | DELETE | STRICT |
| DetectorID | DELETE | STRICT |
| DeviceSerialNumber | DELETE | STRICT |
| DigitalSignaturesSequence | DELETE | STRICT |
| RETIRED_DischargeDiagnosisDescription | DELETE | STRICT |
| RETIRED_DistributionAddress | DELETE | STRICT |
| RETIRED_DistributionName | DELETE | STRICT |
| EthnicGroup | DELETE | STRICT |
| FailedSOPInstanceUIDList | DELETE | STRICT |
| FrameComments | DELETE | STRICT |
| GantryID | DELETE | STRICT |
| GeneratorID | DELETE | STRICT |

Processing Type:

Deleted = Information is removed

Anonymous = Content is replaced with "anonymous"

0 = Content is replaced with "0"

Strict = Second option when installing Local/Anonymization Uplink

Non-strict = Third option when installing Local Anonymization Uplink

| | | |
|---|--------|------------|
| GraphicAnnotationSequence | DELETE | STRICT |
| HumanPerformerName | DELETE | STRICT |
| HumanPerformerOrganization | DELETE | STRICT |
| IconImageSequence | DELETE | STRICT |
| RETIRED_IdentifyingComments | DELETE | STRICT |
| ImageComments | DELETE | STRICT |
| RETIRED_ImagePresentationComments | DELETE | STRICT |
| ImagingServiceRequestComments | DELETE | STRICT |
| RETIRED_Impressions | DELETE | STRICT |
| InstitutionAddress | DELETE | STRICT |
| InstitutionCodeSequence | DELETE | STRICT |
| InstitutionName | DELETE | STRICT |
| InstitutionalDepartmentName | DELETE | STRICT |
| RETIRED_InsurancePlanIdentification | DELETE | STRICT |
| IntendedRecipientsOfResultsIdentificationSequence | DELETE | STRICT |
| RETIRED_InterpretationApproverSequence | DELETE | STRICT |
| RETIRED_InterpretationAuthor | DELETE | STRICT |
| RETIRED_InterpretationDiagnosisDescription | DELETE | STRICT |
| RETIRED_InterpretationDIssuer | DELETE | STRICT |
| RETIRED_InterpretationRecorder | DELETE | STRICT |
| RETIRED_InterpretationText | DELETE | STRICT |
| RETIRED_InterpretationTranscriber | DELETE | STRICT |
| RETIRED_IssuerOfAdmissionID | DELETE | STRICT |
| IssuerOfPatientID | DELETE | STRICT |
| RETIRED_IssuerOfServiceEpisodeID | DELETE | STRICT |
| LastMenstrualDate | DELETE | STRICT |
| MAC | DELETE | STRICT |
| MedicalAlerts | DELETE | STRICT |
| MedicalRecordLocator | DELETE | STRICT |
| MilitaryRank | DELETE | STRICT |
| ModifiedAttributesSequence | DELETE | STRICT |
| RETIRED_ModifiedImageDescription | DELETE | STRICT |
| RETIRED_ModifyingDeviceID | DELETE | STRICT |
| RETIRED_ModifyingDeviceManufacturer | DELETE | STRICT |
| NamesOfIntendedRecipientsOfResults | DELETE | STRICT |
| NameOfPhysiciansReadingStudy | DELETE | STRICT |
| Occupation | DELETE | STRICT |
| OperatorsName | DELETE | NOT_STRICT |

Processing Type:

Deleted = Information is removed

Anonymous = Content is replaced with "anonymous"

0 = Content is replaced with "0"

Strict = Second option when installing Local/Anonymization Uplink

Non-strict = Third option when installing Local Anonymization Uplink

| | | |
|--|--------|------------|
| OperatorIdentificationSequence | DELETE | STRICT |
| OriginalAttributesSequence | DELETE | STRICT |
| OrderCallbackPhoneNumber | DELETE | STRICT |
| OrderEnteredBy | DELETE | STRICT |
| OrderEntererLocation | DELETE | STRICT |
| OtherPatientIDs | DELETE | STRICT |
| OtherPatientIDsSequence | DELETE | STRICT |
| OtherPatientNames | DELETE | STRICT |
| RETIRED_OverlayComments | DELETE | STRICT |
| OverlayData | DELETE | STRICT |
| RETIRED_OverlayDate | DELETE | STRICT |
| RETIRED_OverlayTime | DELETE | STRICT |
| ParticipantSequence | DELETE | STRICT |
| PatientAddress | DELETE | NOT_STRICT |
| PatientComments | DELETE | STRICT |
| PatientSexNeutered | DELETE | STRICT |
| PatientState | DELETE | STRICT |
| PatientAge | DELETE | STRICT |
| PatientBirthTime | DELETE | STRICT |
| PatientInstitutionResidence | DELETE | STRICT |
| PatientInsurancePlanCodeSequence | DELETE | STRICT |
| PatientMotherBirthName | DELETE | STRICT |
| PatientPrimaryLanguageCodeSequence | DELETE | STRICT |
| PatientPrimaryLanguageModifierCodeSequence | DELETE | STRICT |
| PatientReligiousPreference | DELETE | STRICT |
| PatientSize | DELETE | STRICT |
| PatientTelephoneNumbers | DELETE | STRICT |
| PatientWeight | DELETE | STRICT |
| PerformingPhysicianName | DELETE | STRICT |
| PhysiciansReadingStudyIdentificationSequence | DELETE | STRICT |
| PhysiciansOfRecordIdentificationSequence | DELETE | STRICT |
| PreMedication | DELETE | STRICT |
| ReferringPhysicianAddress | DELETE | NOT_STRICT |
| ReferringPhysicianIdentificationSequence | DELETE | STRICT |
| ReferringPhysicianTelephoneNumbers | DELETE | NOT_STRICT |
| PatientTransportArrangements | DELETE | STRICT |
| PerformedLocation | DELETE | STRICT |
| PerformedProcedureStepDescription | DELETE | STRICT |

Processing Type:

Deleted = Information is removed

Anonymous = Content is replaced with "anonymous"

0 = Content is replaced with "0"

Strict = Second option when installing Local/Anonymization Uplink

Non-strict = Third option when installing Local Anonymization Uplink

| | | |
|--|--------|--------|
| PerformedProcedureStepEndDate | DELETE | STRICT |
| PerformedProcedureStepEndTime | DELETE | STRICT |
| PerformedProcedureStepID | DELETE | STRICT |
| PerformedProcedureStepStartDate | DELETE | STRICT |
| PerformedProcedureStepStartTime | DELETE | STRICT |
| PerformedStationAETitle | DELETE | STRICT |
| PerformedStationGeographicLocationCodeSequence | DELETE | STRICT |
| PerformedStationName | DELETE | STRICT |
| PerformedStationNameCodeSequence | DELETE | STRICT |
| PerformingPhysicianIdentificationSequence | DELETE | STRICT |
| PersonAddress | DELETE | STRICT |
| PersonTelephoneNumbers | DELETE | STRICT |
| RETIRED_PhysicianApprovingInterpretation | DELETE | STRICT |
| PlateID | DELETE | STRICT |
| PregnancyStatus | DELETE | STRICT |
| ProtocolName | DELETE | STRICT |
| RETIRED_ReasonForTheImagingServiceRequest | DELETE | STRICT |
| RETIRED_ReasonForStudy | DELETE | STRICT |
| ReferencedDigitalSignatureSequence | DELETE | STRICT |
| ReferencedImageSequence | DELETE | STRICT |
| ReferencedPatientAliasSequence | DELETE | STRICT |
| ReferencedPatientSequence | DELETE | STRICT |
| ReferencedPerformedProcedureStepSequence | DELETE | STRICT |
| ReferencedSOPInstanceMACSequence | DELETE | STRICT |
| RequestingPhysician | DELETE | STRICT |
| RequestingService | DELETE | STRICT |
| ResponsibleOrganization | DELETE | STRICT |
| ResponsiblePerson | DELETE | STRICT |
| RETIRED_ResultsComments | DELETE | STRICT |
| RETIRED_ResultsDistributionListSequence | DELETE | STRICT |
| RETIRED_ResultsIDIssuer | DELETE | STRICT |
| ReviewerName | DELETE | STRICT |
| ScheduledHumanPerformersSequence | DELETE | STRICT |
| RETIRED_ScheduledPatientInstitutionResidence | DELETE | STRICT |
| ScheduledPerformingPhysicianIdentificationSequence | DELETE | STRICT |
| ScheduledPerformingPhysicianName | DELETE | STRICT |
| ScheduledProcedureStepEndDate | DELETE | STRICT |
| ScheduledProcedureStepEndTime | DELETE | STRICT |

Strict = Second option when installing Local/Anonymization Uplink
 Non-strict = Third option when installing Local Anonymization Uplink

| | | |
|--|--------------|------------|
| ScheduledProcedureStepDescription | DELETE | STRICT |
| ScheduledProcedureStepLocation | DELETE | STRICT |
| ScheduledProcedureStepStartDate | DELETE | STRICT |
| ScheduledProcedureStepStartTime | DELETE | STRICT |
| ScheduledStationAETitle | DELETE | STRICT |
| ScheduledStationGeographicLocationCodeSequence | DELETE | STRICT |
| ScheduledStationName | DELETE | STRICT |
| ScheduledStationNameCodeSequence | DELETE | STRICT |
| RETIRED_ScheduledStudyLocation | DELETE | STRICT |
| RETIRED_ScheduledStudyLocationAETitle | DELETE | STRICT |
| SeriesDescription | DELETE | NOT_STRICT |
| ServiceEpisodeDescription | DELETE | STRICT |
| ServiceEpisodeID | DELETE | STRICT |
| SmokingStatus | DELETE | STRICT |
| SourceImageSequence | DELETE | STRICT |
| SpecialNeeds | DELETE | STRICT |
| StationName | DELETE | STRICT |
| RETIRED_StudyComments | DELETE | STRICT |
| StudyDescription | DELETE | NOT_STRICT |
| StudyID | DELETE | STRICT |
| RETIRED_StudyIDIssuer | DELETE | STRICT |
| RETIRED_TextComments | DELETE | STRICT |
| TextString | DELETE | STRICT |
| TimezoneOffsetFromUTC | DELETE | STRICT |
| RETIRED_TopicAuthor | DELETE | STRICT |
| RETIRED_TopicKeywords | DELETE | STRICT |
| RETIRED_TopicSubject | DELETE | STRICT |
| RETIRED_TopicTitle | DELETE | STRICT |
| AffectedSOPInstanceUID | DELETE | STRICT |
| DigitalSignatureUID | DELETE | STRICT |
| ConcatenationUID | GENERATE_UID | STRICT |
| ContextGroupExtensionCreatorUID | GENERATE_UID | STRICT |
| CreatorVersionUID | GENERATE_UID | STRICT |
| DeviceUID | GENERATE_UID | STRICT |
| DimensionOrganizationUID | GENERATE_UID | STRICT |
| DoseReferenceUID | GENERATE_UID | STRICT |
| TransactionUID | GENERATE_UID | STRICT |
| UID | GENERATE_UID | STRICT |

Processing Type:

Deleted = Information is removed

Anonymous = Content is replaced with "anonymous"

0 = Content is replaced with "0"

Strict = Second option when installing Local/Anonymization Uplink

Non-strict = Third option when installing Local Anonymization Uplink

| | | |
|--|-----------------------------------|------------|
| FiducialUID | GENERATE_UID | STRICT |
| FrameOfReferenceUID | GENERATE_UID | STRICT |
| InstanceCreatorUID | GENERATE_UID | STRICT |
| IrradiationEventUID | GENERATE_UID | STRICT |
| RETIRED_LargePaletteColorLookupTableUID | GENERATE_UID | STRICT |
| MediaStorageSOPInstanceUID | GENERATE_UID | STRICT |
| PaletteColorLookupTableUID | GENERATE_UID | STRICT |
| ReferencedFrameOfReferenceUID | GENERATE_UID | STRICT |
| RETIRED_ReferencedGeneralPurposeScheduledProcedureStepTransactionUID | GENERATE_UID | STRICT |
| ReferencedSOPInstanceUID | GENERATE_UID | STRICT |
| ReferencedSOPInstanceUIDInFile | GENERATE_UID | STRICT |
| RETIRED_RelatedFrameOfReferenceUID | GENERATE_UID | STRICT |
| RequestedSOPInstanceUID | GENERATE_UID | STRICT |
| SeriesInstanceUID | GENERATE_UID | STRICT |
| SOPInstanceUID | GENERATE_UID | STRICT |
| StudyInstanceUID | GENERATE_UID | STRICT |
| SynchronizationFrameOfReferenceUID | GENERATE_UID | STRICT |
| RETIRED_TemplateExtensionCreatorUID | GENERATE_UID | STRICT |
| RETIRED_TemplateExtensionOrganizationUID | GENERATE_UID | STRICT |
| ReferencedStudySequence | DELETE | STRICT |
| RegionOfResidence | DELETE | STRICT |
| RequestAttributesSequence | DELETE | STRICT |
| RequestedContrastAgent | DELETE | STRICT |
| RequestedProcedureComments | DELETE | STRICT |
| RequestedProcedureDescription | DELETE | STRICT |
| RequestedProcedureID | DELETE | STRICT |
| RequestedProcedureLocation | DELETE | STRICT |
| VerifyingObserverSequence | DELETE | STRICT |
| VerifyingOrganization | DELETE | STRICT |
| VisitComments | DELETE | STRICT |
| PatientIdentityRemoved | DELETE | STRICT |
| DeidentificationMethod | DELETE | STRICT |
| VerifyingObserverName | ANONYMOUS | STRICT |
| PatientName | ANONYMOUS | STRICT |
| PatientSex | 0 | NOT_STRICT |
| PatientBirthName | ANONYMOUS | STRICT |
| AcquisitionDateTime | DATE_PRESERVE_YEAR_ONLY+NULL_TIME | NOT_STRICT |
| AcquisitionDate | DATE_PRESERVE_YEAR_ONLY | NOT_STRICT |

Processing Type:

Deleted = Information is removed

Anonymous = Content is replaced with "anonymous"

0 = Content is replaced with "0"

Strict = Second option when installing Local/Anonymization Uplink

Non-strict = Third option when installing Local Anonymization Uplink

| | | |
|---|---------------------------|------------|
| AcquisitionTime | NULL_TIME | STRICT |
| InstanceCreationDate | DATE_PRESERVE_YEAR_ONLY | NOT_STRICT |
| InstanceCreationTime | NULL_TIME | STRICT |
| AdmittingDate | DATE_PRESERVE_YEAR_ONLY | STRICT |
| AdmittingTime | NULL_TIME | STRICT |
| StudyDate | DATE_PRESERVE_YEAR_ONLY | NOT_STRICT |
| StudyTime | NULL_TIME | STRICT |
| SeriesDate | DATE_PRESERVE_YEAR_ONLY | NOT_STRICT |
| SeriesTime | NULL_TIME | STRICT |
| ContentDate | DATE_PRESERVE_YEAR_ONLY | NOT_STRICT |
| ContentTime | NULL_TIME | STRICT |
| PersonName | ANONYMOUS | STRICT |
| ReferringPhysicianName | ANONYMOUS | NOT_STRICT |
| PlacerOrderNumberImagingServiceRequest | 0 | STRICT |
| PatientBirthDate | DATE_PRESERVE_DECADE_ONLY | STRICT |
| PatientID | NULL_TIME | STRICT |
| AccessionNumber | "0" | STRICT |
| ContentCreatorName | ANONYMOUS | STRICT |
| ContrastBolusAgent | XX | STRICT |
| FillerOrderNumberImagingServiceRequest | 0 | STRICT |
| RETIRED_FillerOrderNumberImagingServiceRequestRetired | 0 | STRICT |
| PersonIdentificationCodeSequence | 0 | STRICT |
| VerifyingObserverIdentificationCodeSequence | 0 | STRICT |
| DcmTag(0x6301,0x1010,KRETZ_PRIVATE) | DELETE | NOT_STRICT |
| DcmTag(0x6301,0x1011, KRETZ_PRIVATE) | DELETE | NOT_STRICT |
| DcmTag(0x6301,0x1012,KRETZ_PRIVATE) | DELETE | NOT_STRICT |
| DcmTag(0x6301,0x1013,KRETZ_PRIVATE) | DELETE | NOT_STRICT |
| DcmTag(0x6301,0x1022,KRETZ_PRIVATE) | DELETE | NOT_STRICT |
| All Other Private Tags | DELETE | STRICT |