



Using the Viewer

The Tricefy Viewer provides a colorful and up-close look at all your images, clips, reports, and attachments. Open a study or thumbnail in the Viewer by:

- Double clicking a study or thumbnail on the Consults or Studies screens; or
- Single clicking a study on the Studies and selecting the **Open in Viewer** button at the top of the screen;
- Accepting an invitation to a consult (the shared study will open in the Viewer)

Navigating the Viewer

The top of the Viewer contains most of the functionality:

- The   buttons scroll to the next and previous studies on your Studies screen
- Thumbnails selected on the Studies screen remain selected in the viewer. This means that if you share the study with a patient or colleague, they will only receive the selected images. Selected thumbnails will have a **green** outline.
 - To select or unselect specific thumbnails, return to your Studies screen
 - To select all images within the study, choose **Select All** from the action menu at the top of the screen.

Screen Display

The monitor icon at the top of the screen specifies the number of images to display, where to display them, and toggles the thumbnails sidebar:

Highlight the number of quadrants to display (horizontal and vertical)

- Highlight just one box to view a single image

Select one of the options (shown beneath the quadrants) to determine where the thumbnails display on the screen

- The active option is **blue**; select the active option to disable thumbnails