

Instructions for Use

Tricefy[®] and Tricefy Uplink[™]
Software Version 4

Intended Use:

Tricefy is intended to provide physicians with functionalities for receiving, transmitting, presenting, sharing, communicating, retrieving, and viewing medical images.

Precautions for Use & Specifications:

- Read the Instructions for Use to ensure proper use of your software
- Refer to <https://trice.statuscast.com/#/> for weekly updates, bug fixes, and notifications of any issues
- This software should only be used by qualified and authorized account members
- This software should not be used for making medical diagnosis
- Trice Imaging is not allowed to access patient data or re-send images to patients on behalf of customers
- All data and measurements are pulled directly from the ultrasound system, the Tricefy software does not perform measurements or calculations
- Tricefy Uplink requires a computer with a static IP address, at least 40 MB of disk space, and either Windows 10 or Mac OS X 10.6 or higher
- The computer/server running the Uplink software must be turned on and always connected to the internet
- This software relies on an internet connection; disruption in service or slow internet will affect Tricefy performance

Contact:

Email: support@triceimaging.com

Phone:

- US Phone: 858.397.5216
- UK Phone: +44 1767 512021
- Sweden Phone: +46 10 888 68 30
- Billing: +1 858.397.5217

Contact Support for help with installation, configuration, settings, and any other questions. We are here to assist you.

	Trice Imaging, Inc. 1343 Stratford Ct Del Mar, California, 92014 USA	European Representative Address:	Emergo Europe Prinsessegracht 20 2514 AP Den Haag Netherlands
Tricefy AB Birger Jarlsgatan 57c Stockholm 114 34 Sweden			
Basic UDI-DI: 0860006627145W		Single Registration Number (SRN): US-MF-000010257	
Data Protection Officer: Kris Kumler privacy@triceimaging.com			
Report serious incidents to Trice Imaging and the competent authority of the Member State of the user/patient			

